

Hi-Low (MPS) Seating System

Height Adjustable Seating with Built-in Growth

NOW... More seat cushion choices from Early Intervention to Adult!

- Affordable Customized Seating
- Tilt-in-Space with Child in Seat
- Floor to Table Height Adjustment
- Contoured Cushion Support
- Tool Free Adjustments
- Ergo Harness Design
- Soft-Touch® Material Latex-Free

5 year warranty

See page 7 for warranty details



Optional Angle Adjustable Tray Shown

Easy lock 3" swivel casters.



Features & Accessories



1 Ergonomic Push Handle
Comfortable grip for all heights.

2 Wood Accents
Added structure with a modern design.

3 Tilt-in-Space
-30° to 5° range of tilt adjustment with child in seat.

4 Tool Free
Height adjustment of all cushions.

5 Footrest Angle, Depth and Height Adjustment
Allows for optimal positioning.

6 Swivel Locking Casters
All 4 easy lock 3" swivel casters for safety and control.

7 MPS Tray
79060159 Dark Gray, Small
79060359 Dark Gray, Large
Sold separately.

8 Hi-Low Adjustment
Raise and lower with child in seat.

9 Colorful Contoured Comfort
Vibrant color choices and many cushion sizes available.

Customize Your Size



Small Shell (MS)

Shells

Choose a Shell size to determine your options



Large Shell (ML)

Options

Headrests Cushions for both Small (MS) and Large Shells (ML)

MHS (Standard)



7.5"W x 6.5"H

MHL (w/ Laterals)



7"W x 6"H

MBH (Basic)



7.5"W x 6.5"H
1.5"D

MBL (Basic w/ Laterals)



7"W x 6"H
2.5"D

Back Cushions for Small Shell (MS)

MS2



8"W x 8.5"H

MS4



9"W x 11"H

ML2



10"W x 11"H

Back Cushions for Large Shell (ML)

ML2



10"W x 11"H

ML4



11"W x 13"H

ML6



13"W x 13"H

Seat Cushions for Small Shell (MS)

MS1




7"W x 6"D

MS3



9"W x 8"D

MS5

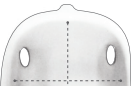

10"W x 10"D

MS7



10"W x 12"D

Seat Cushions for Large Shell (ML)

ML1



11"W x 11"D

ML3



12"W x 13"D

ML5



13"W x 15"D

ML7



13"W x 17"D

Customize Your Size

Fill in your custom sizes here for ease when ordering

MS

Example

MS

Shell

MHS

Head

MS2

Back

MS1

Seat

	Small MPS Seat Cushions				Large MPS Seat Cushions			
MPS Seat Specifications	MS1	MS3	MS5	MS7	ML1	ML3	ML5	ML7
Target User Age	9 mos - 3 yrs	2 - 5 yrs	4 - 8 yrs	5 - 9 yrs	6 - 9 yrs	8 - 11 yrs	10 - 14 yrs	12 yrs - Adult
Target User Height	28" - 37"	32" - 43"	40" - 50"	43" - 52"	45" - 58"	48" - 60"	48" - 62"	58" - Adult
Maximum User Wt.	80 lbs.				130 lbs.			
Seat Width	7"	9"	10"	10"	11"	12"	13"	13"
Seat Depth	6"	8"	10"	12"	11"	13"	15"	17"
Seat to Footrest	6" - 16"				8" - 18"			
Hip Flex Angle	-15° to +15°				-15° to +15°			
Seat Tilt Angle Forward	5°				5°			
Seat Tilt Angle Back	-30°				-30°			
Floor to Seat Ht.	14" - 28.5"				17" - 28.5"			

	Small MPS Back Cushions		Large MPS Back Cushions		
MPS Back Specifications	MS2	MS4	ML2	ML4	ML6
Target User Age	9 mos - 3 yrs	2 - 9 yrs	6 - 9 yrs	8 yrs - Adult	10yrs - Adult
Target User Height	28" - 37"	32" - 52"	45" - 58"	48" - Adult	48" - Adult
Maximum User Wt.	80 lbs.		130 lbs.		
Back Width	8"	9"	10"	11"	13"
Back Height	8.5"	11"	11"	13"	13"
Top of Back Cushion	11" - 16"	15" - 20"	15" - 20"	19" - 22"	19" - 22"
Top of Head Cushion	18" - 23"	21" - 26"	24" - 28"	26" - 30"	26" - 30"

MPS Tray	Small (79060159)	Large (79060359)
Dimensions	24"W x 17"D x 2"H	
Weight	8 lbs.	

MPS Hi-Low Seating System

79300109 Dark Gray, Small
79300309 Dark Gray, Large
Includes Seat Shell with
Footrest, 8-point Harness,
Hi-Low Base with Casters
(Must Choose Custom Seating
Cushions Separately)



Seat Cushions	Cherry	Lilac		Aqua	Dark Gray	
MS1	79050551	79050552		79050554	79050559	
MS3	79050151	79050152		79050154	79050159	
MS5	79050251	79050252		79050254	79050259	
MS7	79050651	79050652		79050654	79050659	
ML1	79050751	79050752		79050754	79050759	
ML3	79050351	79050352		79050354	79050359	
ML5	79050451	79050452		79050454	79050459	
ML7	79050851	79050852		79050854	79050859	

Back Cushions	Cherry	Lilac		Aqua	Dark Gray	
MS2	79120551	79120552		79120554	79120559	
MS4	79120151	79120152		79120154	79120159	
ML2*	79120751	79120752		79120754	79120759	
ML4	79120351	79120352		79120354	79120359	
ML6	79120451	79120452		79120454	79120459	

Headrests	Cherry	Lilac		Aqua	Dark Gray	
MHS (Standard)	79100051	79100052		79100054	79100059	
MHL (w/Laterals)	79110051	79110052		79110054	79110059	
MBH (Basic)	79130051	79130052		79130054	79130059	
MBL (Basic w/ Laterals)	79140051	79140052		79140054	79140059	

*ML2 designed to fit both the Small and Large system, providing a wider Back Cushion option for Small size systems.
Warranty: 5-Year Manufacturer's Warranty on cushions, and seat shell; 2-Year Manufacturer's Warranty on Hi-Low base, and strapping.